
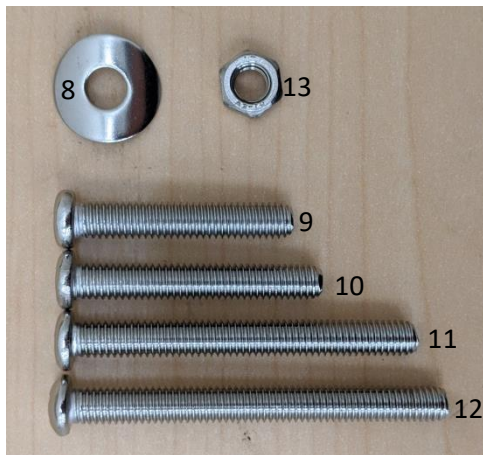


BEMIS Bathtub Transfer Bench Assembly Directions

Parts List:

Item	Description	Quantity
1	Seat	3
2	Back Support	1
3	Arm Rest	1
4	Side Frame Support	2
5	Front/Back Frame Support	2
6	Leg	4
7	Back support tube	2
8	Washer	18
9	M6*35mm (1-3/8")	4
10	M6*40mm (1-1/2")	2
11	M6*55mm (2-3/16")	4
12	M6*60mm (2-3/8")	8
13	Nut	8

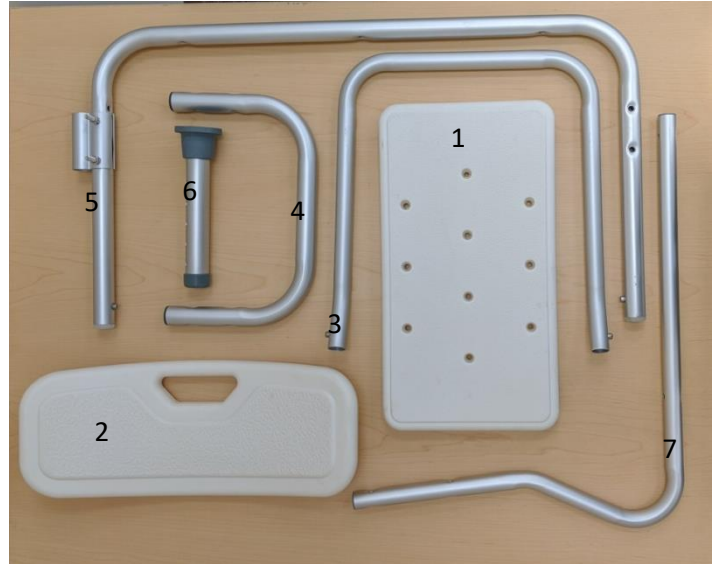

Instructions:

- 1) Attach side frame supports to front/back supports with 4 M6*60mm screws, washers, and nuts on each side. Place the washer under the bolt head and insert thru the holes.


* Make sure arm rest bracket holes are facing outward.



- 2) Attach legs and arm rest by pressing the button and sliding on each part in their given position, make sure buttons secure into the tubes holes.



- 3) Install first seat section to frame supports with a washer and 2 M6*40mm screw. For the next two seats, place back support under the bottom support and place the seat on top, tighten with washers and M6*55mm screws

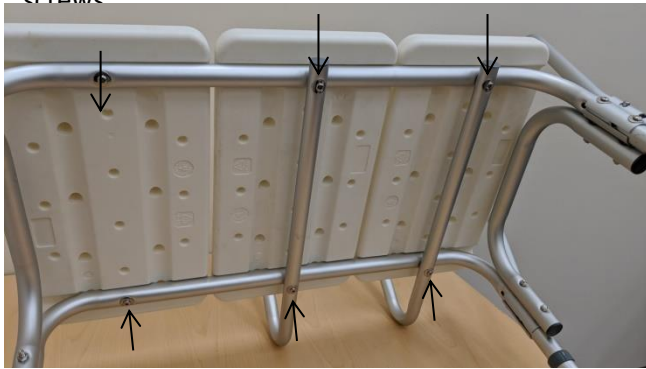

- 4) Attach back support to back support tube with 4 washers and M6*35mm screws.



Assembled Product:



Customer Service: 1-800-558-7651

Bemis Manufacturing Company

Sheboygan Falls, Wisconsin, USA